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EN - Instruction to use
Alkohof TEST is a disposable breathalyzer used to measure the concentration of alcohol in the exhaled breath. Alkohol TEST detects the presence of alcohol at the 0.2 %, 0.5 %, and 0.8 % BAC levels (see the color scale on the tube and package). Alkohol TEST detects the presence of alcohol at the 0.2 %, 0.5 %, and 0.8 % BAC levels (see the color scale on the tube and package). Alkohol TEST detects the presence of alcohol at low as 0.2 % of bood alcohol concentration (BAC), and distinguishes low levels of alcohol from levels in excess of 0.1 mg/l in the exhaled breath, Already 0.2 % BAC impairs driving skills and increase the risk of being involved in a crash.

Important to note: BAC evel rises during an hour after last alcohol consumption. BAC will also depend on a number of other factors including: sex, age, physical condition, amount of food or drugs consumed, type of alcohol as well as many other factors. Alkohol TEST contains a glass tube filled with active substance in the adsorption granules form, filters, foil and 2 protection caps. Test Performance:

1. Walt 15 minutes after last alcohol consumption and do not smoke before performing the test, 2. Push the two ends of the tube by pressing strongly to pierce the protective foil on both sides of the tube. 3. Take a deep breath and blow into the mouthpiece of the tubes the crystals inside the roystals inside the tube become warm. If this does not happen, go back to the step 2. 4. Read errors and the side of the tube. Back to not happen, go back to the step 2. 4. Read errors and the side of the tube. The crystals inside the tube become warm. If this does not happen, go back to the step 2. 4. Read errors and the side of the tube. The crystals inside the tube become warm. If this does not happen, go back to the step 2. 4. Read errors and the altohol to the crystals

manufacturer is not regardly responsible to the results of test reliability, -Manufacturer and dealers are not legally responsible for not intended use or failure to use this instruction by the subject			
ш	Hersteller Manufacturer	2	Nicht wiederverwenden Do not reuse
(]i	Gebrauchsanweisung beachten Follow instructions	<u>₹</u>	Inhalt ausreichend für 1 Prüfung Content sufficient for 1 test
se X we	Bei 4-30°C trocken lagern Store at 4-30°C	NanoRepro AG Untergasse 8 35037 Marburg, Germany	