1 Test zur Eigenanwendung
Zuhause

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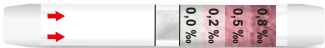
EN - Wait at least 15 minutes after last alcohol consumption.

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EN - Push the two ends of the tube by pressing strongly to break the security foil on both sides of the tube.

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EN - Take a deep breath and blow into the mouthpiece following the arrows on the tube in a continuous breath 2 x 10 seconds. Do not inhale while blowing into the detector.

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
EN - Read results at 2-4 minutes by identifying the colour of the crystals to determine the relative BAC. Do not read results after 5 minutes.

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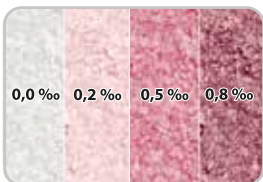EN - Instruction to use

ZuhauseTEST Alkohol is a disposable breathalyzer used to measure the concentration of alcohol in the exhaled breath. ZuhauseTEST Alkohol detects the presence of alcohol at the 0.2 ‰, 0.5 ‰ and 0.8 ‰ BAC levels (see the color scale on the tube and package). ZuhauseTEST Alkohol detects the presence of alcohol as low as 0.2 ‰ of blood alcohol concentration (BAC), and distinguishes low levels of alcohol from levels in excess of 0.1 mg/l in the exhaled breath. Already 0.2 ‰ BAC impairs driving skills and increase the risk of being involved in a crash.

Important to note: BAC level rises during an hour after last alcohol consumption. BAC will also depend on a number of other factors including: sex, age, physical condition, amount of food or drugs consumed, type of alcohol as well as many other factors. ZuhauseTEST Alkohol contains a glass tube filled with active substance in the adsorption granules form, filters, foil and 2 protection caps.

Test Performance:

1. Wait 15 minutes after last alcohol consumption and do not smoke before performing the test. Do not perform the test at temperatures below 2°C or above 30°C. **2.** Push the two ends of the tube by pressing strongly to pierce the protective foil on both sides of the tube. **3.** Take a deep breath and blow into the mouthpiece of the test following the arrows on the tube in a continuous breath 2 times for 10 seconds. Do not inhale while blowing into the detector. Make sure air is coming out from the other side of the tube. The crystals inside the tube become warm. If this does not happen, go back to the step 2. **4.** Read results at 2 minutes by identifying the color of the crystals to determine the relative BAC. Do not read results after 5 minutes. **5.** White crystals inside the tube will change color in the presence of alcohol in the exhaled breath. The relative BAC results may be read by comparing with the color scale on the tube or packaging. Please note that the color change may have different intensity. Some white areas may still remain even though the presence of alcohol is shown on the test. **6.** Dispose the test following local regulations.

Precautions:

-Open ZuhauseTEST Alkohol directly before use, -Do not reuse the ZuhauseTEST Alkohol, -Keep ZuhauseTEST Alkohol away from direct sunlight, high and low temperatures, -Store ZuhauseTEST Alkohol in a dry and safe place, -Keep out of reach of children, -Particular components of ZuhauseTEST Alkohol may pose a risk to life and health. Incorrect use may lead to the glass tube breaking or shattering, causing cuts and injuries. There is a swallow risk of the elements, -ZuhauseTEST Alkohol results may not be reliable if testing is not conducted according to these instructions, -Do not use after the expiration date, -ZuhauseTEST Alkohol is designed for personal use only and should be taken as a warning that the subject may have detectable alcohol in their system. It is not the official BAC measurement test. A positive result should be confirmed by an evidentiary alcohol test, -This is not the diagnostic medical device, -Decisions and/or actions based on the use of this test by any person shall be at person's own risk. -ZuhauseTEST Alkohol provides only a preliminary result for the relative BAC. The manufacturer is not legally responsible for the results or test reliability, -Manufacturer and dealers are not legally responsible for not intended use or failure to use this instruction by the subject

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NanoRepro AG